

Patient Information

MucoMatrixX[®]

BoneProtect[®] Guide

BoneProtect[®] Membrane

BoneProtect[®] Fleece

BoneProtect[®] Cone

Dear Sir or Madam,

Your attending doctor recommends the use of a BoneProtect® product or a MucoMatrixX.

This patient information shall support the understanding of the treatment and the used materials.

Why is a BoneProtect® product/ MucoMatrixX® required?

For you a dental treatment is planned, which includes tissue replacement, tissue enforcement or tissue coverage. For example, to achieve an incident-free healing process, it could be necessary to cover bone graft substitutes with a membrane. In case of a periodontological treatment, membranes functions as a barrier to separate the fast-growing gingiva from the treated root of the tooth to enable the slower growing bone to expand and to regenerate in the protected area.

Bone Protect® Cone and BoneProtect® Fleece can be used for haemostyptic reasons and to support the wound healing.

Why does your dentist recommend a BoneProtect® product/ MucoMatrixX®?

All BoneProtect® products and MucoMatrixX® derived from native collagen and are already used for a long time for various medical products, treatments and indications. All soft tissue grafts are very similar to human soft tissue, which causes a high biologic compatibility and tolerance and leads to a guided and predictable healing process.

The primary material collagen is treated in a multi-step process. This process provides safety and antigenicity of the resulting material, but also preserves the advantageous characteristics of the native collagen.

What happens with the BoneProtect® products/ MucoMatrixX® in your body?

All BoneProtect® products and MucoMatrixX® will be permeated by patients own tissue and resorbed and continuously remodeled by the human body into body-own tissue. An additional operation for removal of the tissue is not necessary. The materials will be remodelled after the appro. time below:

- BoneProtect® Cone und BoneProtect® Fleece 2-4 weeks
- BoneProtect® Membrane 3-6 months
- BoneProtect® Guide 2-3 months
- MucoMatrixX® 6-12 months

Are BoneProtect® Products/MucoMatrixX® safe materials?

The primary material collagen is treated in a multi-step process. This process provides safety and antigenicity of the resulting material, but also preserves the advantageous characteristics of the native collagen. All material sourcing and production is done in Germany, supervised, certified, approved and controlled by the relevant German health authorities. During the process, the material and product is subject to rigid quality and safety controls, the resulting soft tissue product fulfills all national and international safety and health standards.

What are the alternatives for BoneProtect® Products/MucoMatrixX®?

Alternatives for the use of collagen based soft tissue grafts are transplantation of patients own soft tissue or the use of synthetic materials. Also available are several membranes of animal origin with different structural and biological properties. Your dentist will inform you about the treatment with BoneProtect® membrane, BoneProtect® collagen fleece, BoneProtect® Cone, BoneProtect® Guide or MucoMatrixX® and possible alternatives.

Patients consent (please cross out the non-relevant products):

I was informed by my dentist about a treatment with MucoMatrixX®, BoneProtect® Guide, BoneProtect® Membrane, BoneProtect® Fleece or BoneProtect® Cone. I have read and understood the content of the information and agree to the treatment with MucoMatrixX®, BoneProtect® Guide, BoneProtect® Membrane, BoneProtect® Fleece and/or BoneProtect® Cone.

Place, Date

Signature of the patient